



LP — 20

HYGIENE

AUSTRIA

Respirator Mask, FFP2

1. Made in Austria
2. Self and third party protection
3. Great comfort
4. High breathability due to high quality non-woven materials



1. Wash your hands using soap and water or a hand sanitizer.



2. Pull the loops of the mask back to fit them around your ears.



3. Pull the upper and lower ends of the mask over nose and chin, so that no open gaps remain between face and mask.



4. Lightly press the metal nose piece in your mask across nose and cheeks to fit.



5. Avoid touching the mask while wearing.



6. Dispose of the mask safely immediately after wear.

Mask Specifications

Brand	- Hygiene Austria LP GmbH
Name	- Respirator Mask, FFP2
Type	- PSA-Category III
Device Category	- FFP2
Quantity/Units	- Singles or 20 pc. per unit
Colour	- White
Instructions and Durability	<ul style="list-style-type: none">- Shelf life: 5 years- Suitable for healthcare professionals- It is recommended to remove and dispose of the mask every 4 hours or as soon as humid.- Non-reusable, disposable
Storage Conditions	- Store above 0°C and below 40°C in a dry place without direct sunlight
Size	- 16x11cm folded (slight deviations possible)
Weight	- 5.25 g (slight deviations possible)
Certificates	<ul style="list-style-type: none">- CPA-FFP2- OETI certified according to GZ 2020-0.247.451
Product specifications	<ul style="list-style-type: none">- Made out of 5 layers of non-woven material, 2 outer layers spunbond nonwovens, 2 inner layers hot-air-cotton and 1 inner layer meltblown nonwovens polypropylene- 9cm extra strong and flexible nose piece made of two metal wires coated in rubber

Please check the authenticity of all certificates offered to you by suppliers through:
<https://www.eu-esf.org/covid-19/4513-covid-19-suspicious-certificates-for-ppe>
Your health will thank you.